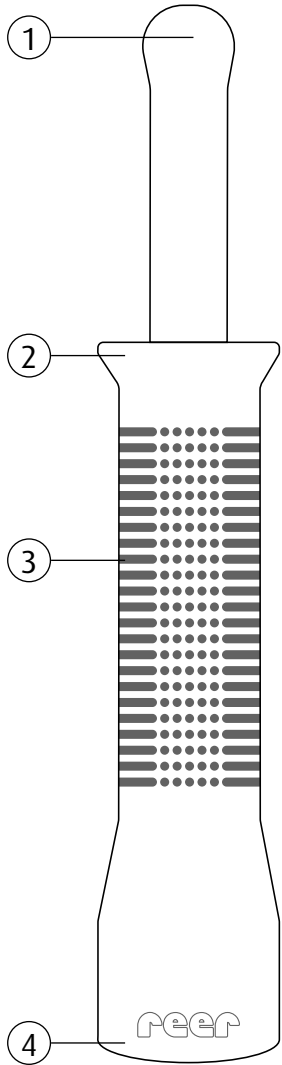
DE Relieve Blähhilfe // Art. Nr.: 48130

GB Relieve Wind reliever // Item no.: 48130

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Instruction manual

GB

IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!

The Relieve wind reliever from reer is a disposable catheter for the removal of intestinal gas in babies and provides relief from flatulence, colic or constipation.

1. NOTE! IMPORTANT SAFETY INSTRUCTIONS

- ! This product is not a toy!
- ! Always keep the product out of the reach of children.
- ! Do not leave children unattended with the product. Only for use by adults.
- ! Check each wind reliever for visible damage before use.
- ! The product must be disposed of after use.
- ! Please only use the device for its intended purpose.

2. Health hazard

- ! Do not leave the product in the child's rectum.
- ! If resistance occurs, pull back and remove carefully.
- ! Re-use of the same wind reliever may pose a risk of infection.
- ! Do not use a wind reliever more than 3 times within 24 hours.
- ! Consult a doctor if redness, irritation or bleeding occurs before, during or after using the wind reliever, as these may be signs of a serious medical condition.
- ! Do not use the wind reliever if your child has a sore or irritated bottom or anus.
- ! Unused wind relievers should be disposed of after a maximum of 3 years.
- ! All serious incidents that have occurred in connection with the product must be reported to the manufacturer or the responsible authority of the Member State in which the user lives.

3. Product description (see graphic)

- | | |
|-----------------------|--------------------|
| 1. soft, flexible tip | 3. textured handle |
| 2. stopper | 4. opening |

4. Use

1. Gently massage your baby's tummy from top to bottom.
2. Have a nappy ready in case of stool leakage.
3. Lubricate the tip of the wind reliever with baby oil.
4. Raise your baby's legs and carefully insert the wind reliever into the anus like a clinical thermometer. If you feel any resistance, carefully pull back the wind reliever and remove it.
5. After a few seconds, you will clearly hear the air escaping from the bowel.
6. Now dispose of the used wind reliever in the residual waste! The wind reliever must only be used once!

5. Warranty

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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