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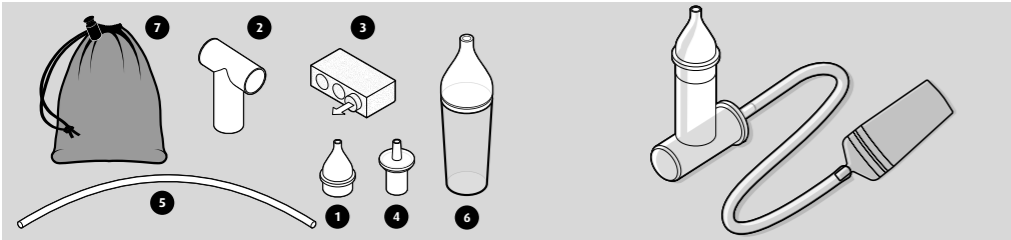
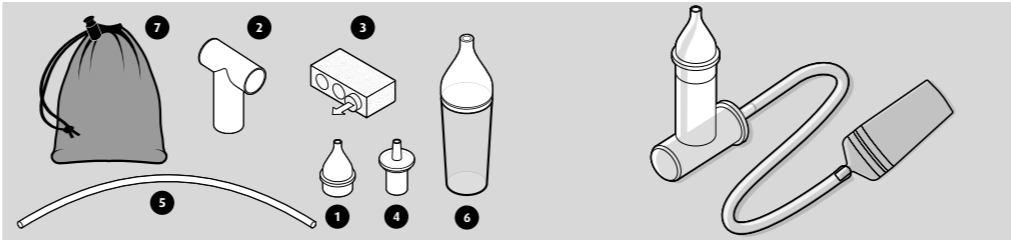
GB Nasal aspirator with vacuum cleaner adapter
Instructions for use // item no.: 79149

IMPORTANT!
THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!

Deliverables and part description

- | | |
|--|---------------------------|
| 1. Soft silicone tip | 4. Hose connector |
| 2. Collecting container | 5. Hose |
| 3. Hygiene filter (3 pcs; can be washed out or steam sterilised) | 6. Vacuum cleaner adaptor |
| | 7. Storage bag |

CAUTION!
If your child is sick, consult a doctor.

Safety instructions

- Please only use the item for its intended purpose.
- Check the article for visible damage before each use. If there are any signs of damage, do not use. Dispose of product!

Risk of injury
Do not use the nasal aspirator for nose bleeds or nostril wounds. Do not use on the mouth or ears.

Choking hazard

- Not a toy, keep out of the reach of children.
- Do not leave children unattended with the product. Only for use by adults.
- Make sure that all parts are connected securely to one another before use.

Health risk
If you do not dry the parts completely before assembly, there is a risk of bacteria and mould growth.

Attention!

- Ensure sufficient clearance between your child and the vacuum cleaner.
- Not following the instructions can endanger your child's health!

INSTRUCTION FOR USE
Before treatment
If nasal secretion is too tough or dry, try to soften the blockage with an isotonic saline solution, spray or drops. In case of doubt or uncertainty, please contact your paediatrician.

Application

1. Insert a new or cleaned filter (3) into the hose connection (4).
2. Connect the hose connection (4) to the collection container (2) and the silicone tip (1).
3. Place the hose (5) onto the hose mount of the collection container.
4. Insert the vacuum cleaner adapter (6) into the end of the hose (5) and connect the adapter to your vacuum cleaner.
5. Turn on the vacuum cleaner on the lowest setting and place the soft silicone tip close to your child's nostril.
Caution: Place the Nasal Aspirator close to the opening of the nostril. Do not push it into the nostril!
6. Suck repeatedly for a few seconds until the nasal discharge ends up in the collection container. Repeat this process on the second nostril.
7. You can remove the hose connection and replace the filter if desired. The filters prevent the transfer of bacteria and are washable. The top is designed to avoid damaging the nasal mucous membrane.

Cleaning

1. Dismantle the Nasal Aspirator into its individual parts directly after each use.
2. Wash all parts (including the filter) immediately and very carefully with warm water and mild detergent.
3. Rinse the clean parts with fresh water.
4. Let them dry out completely.
5. Never place the parts in the storage bag when damp – risk of odours, bacteria and mould!
6. When not in use for long periods, we recommend that you clean the nasal aspirator again and sterilise it before the next use.
7. Clean only in the top basket of the dishwasher or as described in points 2 - 4.
8. Disinfecting only possible in the top basket of the steam steriliser or by boiling for approx. 5 minutes.
9. Do not disinfect in the microwave!
10. Let the nasal aspirator cool down and dry completely after cleaning or sterilisation.

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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