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## GB Instruction manual

Postnatal cooling pad // Item no.: 31114

**IMPORTANT! THE FOLLOWING INSTRUCTIONS MUST BE READ CAREFULLY AND RETAINED FOR FUTURE REFERENCE KEEP INSTRUCTIONS SAFE!**

### General safety instructions:

- Please only use the item for its intended purpose.
- Discontinue use if there are signs of defects.
- Please follow the heating instructions without fail, as otherwise, there is a risk of the gel pack bursting.
- Only use the cooling pad for private use. They are not intended for commercial, medical, or therapeutic use. They are not a substitute for treatment by a doctor. If you have any health concerns, consult your doctor or midwife before using the item.
- Keep the item out of the reach of children.
- Do not use in the event of severe and chronic (skin) conditions.
- Remove gel packs from the hygienic cover before heating or cooling.
- Never place gel packs directly on the skin.
- Allow the gel packs to cool down at room temperature before (re-)heating.

### Action to be taken in the event of direct contact with gel:

If the contents come into contact with the skin or eyes, rinse immediately with water. The gel is not toxic.

### Instructions for use:

- Use the cooling pad for a maximum of 20 minutes each time.
- The hygienic cover is intended for single use only.

### For cold use:

Place the gel pack in the freezer for approx. 1 - 2 hours. Store the gel packs in the freezer for a maximum of five hours.

### For warm use:

-Microwave: Heat the gel packs for about 30 seconds in a 600 W microwave.

-If the gel pack is not warm enough, heat it once again for 10 seconds. **Note: NEVER heat the gel pack too much.**

-Boiling water: The gel pack can also be immersed in boiling water for 4-10 minutes. After four minutes, check the gel pack every minute. Use tongs to remove the pack from the water.

### Cleaning instructions:

**Gel pack:** Wipe with a damp cloth. Hygiene cover is a disposable product and not washable!

### Disposal instructions:

Do not throw the hygiene cover into the toilet. Hygiene cover contains plastic and improper disposal has a negative impact on the environment.

Please retain your receipt, the user manual and item number. Warranty claims or claims from promises of guarantee must be processed by the dealer.

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